RESIPLATRE DENTAIRE SOCKEL

Extremely free-flowing super-hard gypsum for vibration-free model bases

Advantages

- Vibration (shaking) no longer required
- No more bubbles between the pins
- No more smeared models
- Low setting expansion
- Up to 10 models can be based with one mixture
- Absolutely smooth model surface
- High hardness
- Major time savings

Physical data

- End of solidification approx 10 min.
- Resistance to pressure above 50 N/mm²
- Hardness after 24 hours above 150 N/mm²
- Linear Setting expansion after 2 hours below 0,08 % after 24 hour below 0,1 %

Processing

Mixing ratio

powder : dist. Water 100 g : 23 - 25 ml

Processing temperature approx. 23 °C

Scatter powder into the water provided

Mixing duration, mechanical, under vacuum 30 sec. Mixing duration, manual 60 sec.

The initial consistency will liquefy as the stirring duration proceeds giving optimum flow characteristics for pouring.

Processing time greater than 5 min.

Delivery form: white and pink If you have special colour wishes, minimum delivery quantity is 2 t.

Packaging units Paper bags with film lining 25 kg

Usability

In tightly sealed, moisture-proof containers, when stored in a dry location, at least 1 year after production.

Please note

Like all gypsum products, base gypsum is sensitive to humidity. The containers must accordingly be resealed after any product has been removed from them.

If the product is transferred from the original containers, use only moisture-proof packaging, like

- plastic-aluminium film/foil laminates

- low-pressure polyethylene cans with a wall thickness of at least 0,5 mm.

The particulars above are given to the best of our knowledge and after meticulous checking. They correspond to the current state of the art. We guarantee faultless quality for our products, but will accept no liability for processing results, which will as a rule be outside our capacity to influence.

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